



## Kendall™ Sputum Collector and Transport System

### Complete Kit

- Eliminates need to gather additional supplies

### Built-in Funnel

- Simplifies sputum collection

### Built-in 50mL Graduated Centrifuge Tube

- Sample can be collected and processed in the same tube

### Lined Cap

- Minimizes leakage

### Double Barrier Protection

- Container and tube provide double barrier for safe transportation



# Sputum Collection and Transport System



## Order Information

Item Number	Description	Case Quantity
2650SA	Sputum Collector	50

## Patient Instructions



Remove white cap. Do not remove outer plastic label.



Expectorate into funnel.



After specimen is collected, replace white cap tightly on container.

## Nurse and Technologist Instructions



After specimen is collected, remove plastic label from container by tearing perforations.



Unscrew funnel section from the container.



Remove centrifuge tube from the funnel.



Close centrifuge tube with orange cap provided.



Remove white cap, dispose funnel in appropriate manner.



Place labeled centrifuge tube in container, replace white cap tightly. Forward to lab.



COVIDIEN, COVIDIEN with logo, Covidien logo and *positive results for life* are U.S. and/or internationally registered trademarks of Covidien AG. <sup>TM</sup> trademarks of a Covidien company.  
©2009 Covidien.  
H6495 5M 1109

**For Ordering Information Contact CHARTECH INC.**  
**1-877-658-1849 Fax 770-442-2843**  
**sales@chartech.net**